

FootSpa Operating Manual



Congratulations, you have purchased the finest intense LED light therapy device specially designed for your clients'/patients' feet.

IMPORTANT: Read the entire operating manual before operating your **FootSpa**. Keep this manual and all **FootSpa** information handy for future reference.

UNPACKING

After you have opened the box your **FootSpa** came in, check to make sure that the following items were included:

- 1 FootSpa
- 1 Power Cord with Power Supply
- 50 Disposable Paper Foot Rests
- 50 of Consent Forms

If any of the above items were not included, please call our customer service center at **888-807-SKIN**.

SET UP


Once you have removed the **FootSpa** from the box, place it on the floor. Plug power cord into back of unit and plug other end into any standard 110 volt outlet. For use outside the United States, the **FootSpa** may be plugged into any outlet 110 to 240 volt and it may be necessary to use an outlet adapter in order for the **FootSpa** plug to properly fit into the electric receptacle. It is now ready for use.

ADMINISTERING A FootSpa SESSION


Have your client/patient review and sign the Customer Consent Form. Your client/patient should be seated in comfortable position facing the front of the **FootSpa**.

Optional: You can apply a serum, cream or gel of your choice to your client's/patient's feet. The FootSpa is designed to be used without any supplemental applications but using one will only enhance the results.

Turn on the **FootSpa** using the on/off switch located at the back of the **FootSpa**.

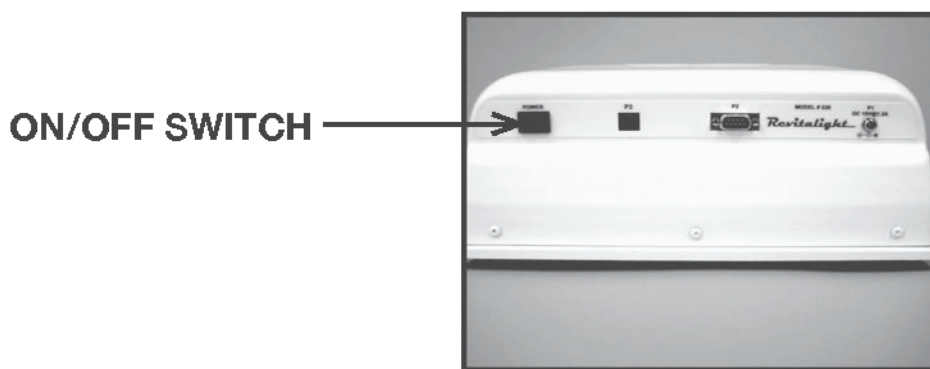

Once the **FootSpa** is on, press the start switch at the top of the unit.

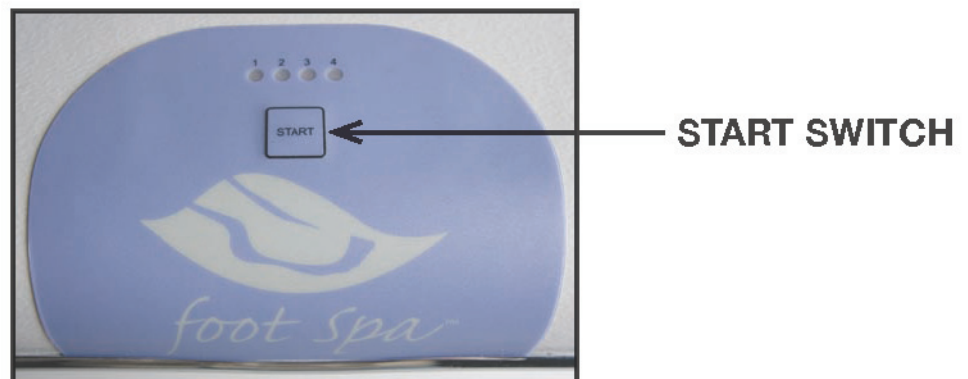

This will turn on the **FootSpa** and automatically set ten (10) minutes on the timer, which is the preset time for a session. Four (4) red indicating LEDs will light on the top of the **FootSpa**. These lights indicate the remaining time of a session. For every 2.5 minutes that has passed, one of the indicating LEDs will go off.

(Example: When only two of the indicating LEDs are lit, there are less than 5 minutes remaining in the session.) Also, when the “Start” button is pressed the intense LEDs will go on. **WARNING! YOU AND/OR YOUR CLIENT/PATIENT SHOULD NEVER LOOK DIRECTLY AT THE LED LIGHTS INSIDE THE FOOTSPA.** At this time, while your client is present, (so they can see that their feet will be resting on a clean surface); insert a new paper foot Rest into the **FootSpa**. This assures your clients’/patients’ feet will be resting on a new/clean surface.

Have your client insert their feet into the **FootSpa**. They should be resting comfortably on the paper insert.



After ten (10) minutes the indicating LED lights and the LED lights in the **FootSpa** will shut off and there will be a tone alerting you that the session has ended. At this time you can instruct your client/patient to remove their feet. The **FootSpa** session is complete.

For feet cosmetic purposes, on average, six (6) FootSpa sessions are recommended, at one (1) week intervals. After the six sessions, a maintenance session every six (6) to eight (8) weeks is recommended.

MAINTENANCE

When needed wipe plastic surfaces with a soft damp cloth. Do not use any chemical or abrasive cleaners.

If you have any questions about operating your **FootSpa**, call our customer service center at **888-807-SKIN**.